



## **Bioclear Blasting Powder Instructions For Use (IFU)**

### **Indications:**

The Bioclear Blasting Powder is indicated for all anterior and posterior restorations where removal of Biofilm is needed. For use with the Bioclear Blaster for cleaning, polishing, and restoring the natural aesthetics of tooth enamel.

### **Contraindications:**

Patients who have severe respiratory illness should consult their physician before undergoing air polishing prophylaxis procedures. Use is not recommended for patients with known allergies to the components of the Blasting Powder.

### **Intended Purpose:**

Bioclear Blasting Powder is intended to be used in conjunction with the Bioclear Blaster to remove Biofilm prior to undergoing a tooth restoration. Blasting Powder is invasive through a body orifice, specifically orally, but does not penetrate past the pharynx.

### **Safety:**

To offer your patients the safest, most comfortable procedure possible, carefully read the enclosed instructions and watch the tutorials on the website prior to use.

Note: Please check [www.bioclearmatrix.com](http://www.bioclearmatrix.com) to find the most recently updated instructions and tutorials.

### **Precautions:**

Bioclear Matrix offers instructions and classes for using these products. Any use not specified in the instructions is not recommended and will be the responsibility of the attending practitioner.

### **Instructions:**

Shake well before using. Use in conjunction with the Bioclear Blaster and in accordance with the instructions for use included with the Bioclear Blaster.

### **Composition:**

See Blasting Powder Safety Data Sheet (SDS). It can be found online at [www.bioclearmatrix.com](http://www.bioclearmatrix.com)

### **Limitation of Liability**

Bioclear Blasting Powder is intended to be used by educated dental professionals, e.g., general dentists, dental assistants, dental hygienists, who have theoretical and practical





knowledge on usage of dental products only. Any use not specified in the instructions is not recommended and will be the responsibility of the attending practitioner.



**Manufacturer:**

Bioclear Matrix Systems, LLC  
3402 South 38<sup>th</sup>  
Tacoma, WA 98409  
USA  
Ph: 855-712-5327

**Origin:** Made in the USA

Any serious incident that has occurred in relation to the use of this device should be reported to the manufacturer immediately either by mail at the address above or by email to [jennifer@bioclearmatrix.com](mailto:jennifer@bioclearmatrix.com).

**Importer:**

Find the importer for your country on our website at [www.bioclearmatrix.com](http://www.bioclearmatrix.com)

**IFU Revision Date:**

May 4, 2021

