



## V-WIPE

### Surface Disinfectant Wipe CONTAINS ALCOHOL

**CLASSIC**  
200 LARGE WIPES  
200 x 200mm



**63%**  
ALCOHOL  
CONTENT

**CONTACT**  
60 SEC



**18gsm**  
EVERYDAY  
WIPE

**200**  
LARGE  
WIPES

#### KEY FEATURES

Bossklein V-WIPES are a powerful, alcohol based disinfectant wet wipe. Designed for the disinfection of non-invasive medical device surfaces within medical and dental environments. Ideal to use where time is critical and also in high traffic areas.

- Contact time of only 60 seconds
- Wide effective spectrum
- Kills TB & Coronavirus
- Quick drying - leaves no smears or residue
- Dual registration; Medical Device and Biocide
- Fresh lemon aroma

Bossklein V-WIPES are ideal to use on alcohol resistant, non-porous surfaces within the dental surgery. The 60 second contact time makes them perfectly suited where short contact times are important, eg in between patients.

The 63% alcohol based formulation has a powerful microbial action, the perfect solution for controlling infections on resistant surfaces in clinical environments.

V-WIPES have been rigorously tested and proven to be effective against all Bacteria and Yeast. This highly effective formula has been tested to real world EN standards for proven effectiveness including Phase II Step I and II. Effective against Coronavirus, HIV, Hepatitis, Influenza, TB, Adeno and Noro viruses in just 60 seconds.

All V-WIPE Classic tubs are refillable and made from 35% recycled plastic (PCR), saving 31.5g of plastic for each tub. Each time a refill is used it saves 85% plastic compared to using a new tub.

The fresh lemon aroma leaves equipment and rooms reassuringly fresh, clean and ready for the next patient.

#### COMPOSITION

Ethanol (64-17-5) 63% w/w (630g/kg), Aqua, Perfumes (Contains Linalool, Citral, Geraniol, Alpha-Isomethyl ionone).

#### INSTRUCTIONS FOR USE

1. Place refill bag in dispenser tub, 2. Cut open air tight plastic pouch across the top of the pouch, 3. Feed the centre wipe through the hole in the cap and replace, 4. Pull wipe at a 45° angle. Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action for 60 seconds. Allow surface moisture to evaporate. For professional use only.

#### REGISTRATION

Dual registered as a Class IIa medical device for use on both non invasive medical devices and as a general purpose Biocide registered disinfectant. This ensures suitability for use within all types of clinical and non clinical environments.

#### TEST STANDARDS

The following test standards have been achieved: EN 16615, EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.

#### VARIANTS AND PACKAGING

BOS9783 Tub of 200 Classic Wipes x 12 per box  
BOS9784 Refill of 200 Classic Wipes x 12 per box  
BOS9784KIT Kit of Classic Wipes - 6 x Refill + 1 x Tub

#### MICROBIOLOGICAL EFFICACY

MICROORGANISM	TIME
<b>Bactericidal</b>	<b>60 sec</b>
BS EN 13727:2012 + A2:2015	
BS EN 16615:2015	
Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, E. coli, MRSA, VRE	60 sec
<b>Yeasticidal</b>	<b>60 sec</b>
BS EN 13624:2013	
BS EN 16615:2015	
Candida albicans	60 sec
<b>Enveloped viruses</b>	<b>60 sec</b>
BS EN 14476:2013 + A1 2015	
Hepatitis B virus (HBV), Hepatitis C virus (HCV), HIV, Measles virus, Hepatitis Delta virus (HDV), Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella virus, Rabies virus, Poxviridae	60 sec
<b>Non enveloped viruses</b>	<b>60 sec</b>
BS EN 14476:2013 + A1 2015	
Adenovirus 5, Murine norovirus	60 sec
<b>Mycobactericidal</b>	<b>60 sec</b>
BS EN 14348:2005	
Mycobacterium avium	60 sec
<b>Tuberculocidal</b>	<b>60 sec</b>
BS EN 14348:2005	
Mycobacterium terrae	60 sec

Wipes are available in our 'Classic' wipe 18gsm substrate, ideal for economic everyday use. Each wipe measures 200x200mm and each pack contains 200 Classic wipes. Reusable dispenser tubs and economical refill packs are available.



Complete **hygiene** solutions

DirectaDentalGroup  
DIRECTA topdental oszima parkell KOHLER RÖNIGG polydentica  
TrollDental Sendoline physics CONTACTEZ ceramir

Topdental (Products) Ltd  
Unit 1, Holmfield Industrial Estate  
Halifax, West Yorkshire HX2 9TN  
Tel +44 (0)1535 652 750 www.topdental.co.uk