





1

Using the calipers provided, measure the patient's arch width by placing the caliper tabs on the lingual of the 1st molar and 2nd bicuspid.

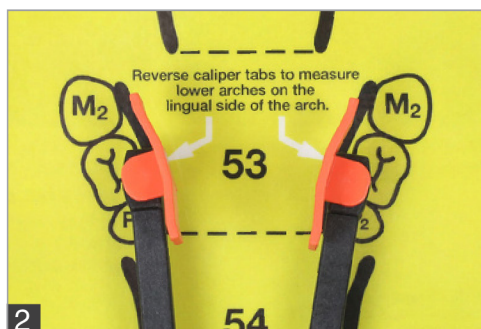

2

Place the calipers on the size chart to determine appropriate tray size.

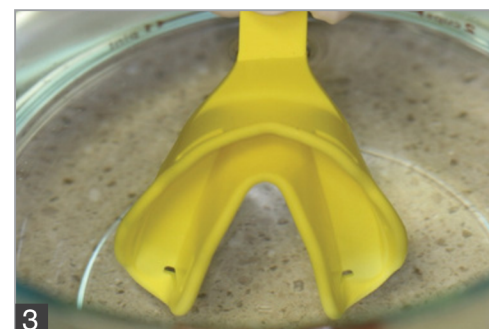

3

Select the relevant or appropriate tray.



4

Immerse selected tray in a water bath of 70oC/158oF for 1 minute.



5

Keep desired form stable for 5 seconds. Check fit in the mouth and repeat the molding procedure if needed. If fit is achieved, run the tray under cold water for 5 seconds to set form.



6

The resulting final impression.

**HEATWAVE INTRO KIT (188932)** Contains: 32 trays (4 each size, upper & lower), measuring caliper & size chart

**HEATWAVE REFILL (10 trays/box)**  
Size 42 Upper Tray (188142) Size 43 Upper Tray (188143) Size 44 Upper Tray (188144) Size 45 Upper Tray (188145) Size 51 Lower Tray (188151) Size 52 Lower Tray (188152) Size 53 Lower Tray (188153) Size 54 Lower Tray (188154)

**HEATWAVE QUADRANT PACK (188710)**  
Contains: 10 trays (5 right trays, 5 left trays)