# CONTACTEZ<sup>®</sup>

*Interproximal Solutions*

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**Composition**

Plastic (polycarbonate)

Diamond plated and non-plated stainless steel

**Precaution**

- To be cleaned and disinfected according to the instructions below prior to first use and between uses. If high-level disinfection not possible or sterilization required by national recommendations, the device shall be sterilized according to the instructions.
- Use minimal vertical force to avoid breakage.

**User instructions**

Use two fingers and carefully pass the strip buccolingually, back and forth with gentle force.

**Examples of use**

*ContactEZ Restorative Strip System:* Proximal contact adjustment and complete marginal seating of crowns, veneers, inlays/onlays as well as proximal contouring of composite fillings.

*ContactEZ IPR Strip System:* Interproximal reduction system.

*ContactEZ Sub Gingival Strip System:* Removal of hard-to-reach subgingival rough margins and overhangs as well as final polishing.

*ContactEZ Hygienist Strip:* Removal of extrinsic discoloration and plaque from interproximal surfaces.

## Maintenance, cleaning and sterilization

|                             |                                                                                                                                                                                                                                                                                              |
|-----------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Product                     | <b>ContactEZ Strips</b> are used to remove and/or finish dental enamel or filling materials                                                                                                                                                                                                  |
| Warnings                    | Place in the middle of the autoclave, not against the autoclave's walls or heating element                                                                                                                                                                                                   |
| Limitations on reprocessing | The characteristics of the plastic are affected by autoclaving, resulting in limited product lifetime. Metal strip may discolor after autoclaving, does not affect safety or performance.                                                                                                    |
| Inspection and maintenance  | All plastic products have a limited-service lifetime and should be replaced at regular intervals. All types of cleaning reduce the product's service lifetime. The product should be replaced when its functioning and/or performance no longer satisfies the specifications in this manual. |

## Instructions

|                             |               |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
|-----------------------------|---------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Preparation before cleaning | Instruction   | Wipe off the instrument after use to prevent soil and debris to dry onto the instrument. Perform cleaning as soon as possible after use. Do not exceed 2 hours.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
| Cleaning:<br>Manual         | Equipment     | Soft bristle brushes of various sizes, Ultrasonic bath                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                   |
|                             | Detergent     | Follow agent manufacturers recommendations for concentration and temperature.<br>Manual: Enzymatic or low-alkaline (pH≤8) detergent suitable for manual cleaning.<br>Ultrasonic bath: Enzymatic or low-alkaline detergent with minimal foaming characteristics.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
|                             | Water quality | Drinking water quality for manual cleaning and distilled or de-mineralized water for ultrasonic bath and final rinse should be used.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                     |
|                             | Instruction   | <ol style="list-style-type: none"><li>1. Immerse in freshly prepared cleaning solution as per manufacturer's instructions although maximum 40°C. Clean mechanically with a brush, working beneath the liquid level, until visibly clean.</li><li>2. Rinse thoroughly</li><li>3. Clean in an ultrasonic bath for a minimum of 5 minutes, using a frequency of 35-45 KHz and power min. 150 W. Make sure the instruments are in an open position.</li><li>4. Rinse thoroughly for min 30 seconds.</li><li>4 a Carefully dry instruments with clean compressed air (Class 1 or better according ISO 8573-1:2010) or lint free wipe. Any remaining water can cause potential rust or corrosion on the instrument surface and into the substrate.</li><li>5. Inspect cleanliness. If necessary, repeat from step 1.</li></ol> |
| Disinfection:<br>Manual     | Detergent     | Immerse into suitable disinfection solution. Follow the manufacturer's instruction of disinfection solution for concentration and time.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                  |

|                                         |                                                                                                                                                                                                                                                                                              |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            |
|-----------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Cleaning and disinfection:<br>Automated | Equipment                                                                                                                                                                                                                                                                                    | Washer-disinfector (validated according to EN ISO 15883)                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                   |
|                                         | Detergent                                                                                                                                                                                                                                                                                    | Enzymatic or low alkaline, suited for medical devices.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                     |
|                                         | Rinsing Agent                                                                                                                                                                                                                                                                                | Non-corrosive, neutral cleaning agent with minimal foaming characteristics, suited for medical devices.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                    |
|                                         | Water quality                                                                                                                                                                                                                                                                                | Follow agent manufacturer's instructions regarding concentration and temperature.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
|                                         | Instructions                                                                                                                                                                                                                                                                                 | Drinking water quality for cleaning and deionized or purified water for final rinse/disinfection.<br><br>1. Load the instruments in an open position in the washer-disinfector. Use a suitable instrument tray.<br>2. Run program:<br>- Pre-wash in cold water, 2 x 2 min<br>- Main wash with detergent at minimum 55°C, 10 min<br>- Rinse in warm water, 2x 1 min.<br>- Final rinse/disinfection in distilled or de-mineralized water at 90°C, minimum 1 min.<br>- Drying at 110° C, minimum 15 min<br>3. When unloading, control that the instruments are clean. If necessary, repeat from step 1 or use manual cleaning |
| Inspection and maintenance              | All plastic products have a limited-service lifetime and should be replaced at regular intervals. All types of cleaning reduce the product's service lifetime. The product should be replaced when its functioning and/or performance no longer satisfies the specifications in this manual. |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            |

|                             |                                                                                                                                                                       |                                                                                                                      |
|-----------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------|
| Packaging for sterilization | Equipment                                                                                                                                                             | Standard packaging pouch                                                                                             |
|                             | Instruction                                                                                                                                                           | Pack the instrument in an open position in the pouch. Place pouches plastic towards plastic and paper towards paper. |
| Sterilization               | Caution                                                                                                                                                               | The instrument must be cleaned and disinfected before sterilization.                                                 |
|                             | Equipment                                                                                                                                                             | Steam autoclave (validated according to EN 13060, EN 285, EN ISO 17665-1)                                            |
|                             | Instructions                                                                                                                                                          | Run minimal cycle:<br>Steam temperature Min. 134 °C / 273°F<br>Steam exposure time 4 min<br>Vacuum drying 3 min      |
| Storage                     | Ensure the instruments are dry before storage. The instrument should be stored in a packed tray, free from dust and moisture at moderate temperatures of 5°C - 40°C . |                                                                                                                      |
| Transportation              | If required, secure transport in closed boxes is recommended in order to avoid any damage to the medical device as well as environmental contamination.               |                                                                                                                      |

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**Directa AB**