

EthOss® Bone Graft Syringe

With a small, curved nozzle, the reusable syringe is suitable for a variety of defects

Offering improved precision in small or difficult-to-access defects, the EthOss® Bone Graft Syringe offers an alternative method of applying graft material to a surgical site.

With a small, curved nozzle, the reusable syringe is suitable for a variety of defects:

- Sinus augmentations, particularly crestal/internal lifts
- Narrow/small surgical sites
- Perio defects, where precision is essential
- Posterior sites with challenging access
- Tunnel grafting



The syringe has been designed to work alongside EthOss® Bone Graft material

EthOss® Bone Graft Syringe Guidelines

Care should be taken over the preparation of the EthOss® Bone Graft material. It may be preferable to mix the powder with saline in stages to prevent it from setting before loading it into the syringe.

1. The EthOss® Bone Graft should be expressed into a sterile receptacle (e.g. dappen dish) without mixing
2. The powder can then be mixed with sterile saline as required, until the material is wetted
3. Using the sterilised Bone Graft Syringe, gather the mixed material into nozzle using a tapping motion
4. Expel the material into the surgical site
5. If additional graft material is required, repeat the process

The EthOss Bone Graft will still require drying in situ using a sterile gauze for 3–5 minutes.

The EthOss® Bone Graft IFU should be consulted and all steps followed.