

**Semi-Gel Etchant  
35% Phosphoric Acid**

**DIRECTIONS FOR USE**

1. Prepare tooth and clean enamel surface to be etched with oil-free prophy paste. Rinse and dry.
2. Dispense etchant into a mixing well or dappen dish. Using a brush, apply to tooth surface. Recommended etching time for enamel is 15-20 seconds and for dentin is 15 seconds.
3. Rinse with copious amounts of water. Do not disturb this surface.
4. Leave surface wet or dry, depending on the instructions for use for the bonding system you are using.

NOTE: Etched enamel should appear chalky white. If not, reapply etch gel to enamel for 20 seconds, rinse with copious amounts of water, and proceed with step #4 above.

WARNING: If etchant contacts skin or soft tissue, rinse immediately with copious amounts of water. Seek medical attention, if required.