Super-Snap Buff Disk

04.0:2020-11
300030540

Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EC REP

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

CE 0123

MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

Super-Snap Buff Disks	10,000 min ⁻¹
-----------------------	--------------------------

en MAXIMUM PERMISSIBLE ROTATION SPEED
de MAXIMAL ZULÄSSIGE DREHZAH
fr VITESSE MAXIMALE DE ROTATION AUTORISÉE
nl MAXIMAAL TOELAATBARE ROTATIESNELHEID
es VELOCIDAD MÁXIMA PERMISIBLE DE ROTACIÓN
it MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA
sv HÖGSTA TILLÅTNA ROTATIONSASTIGHET
pt VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

en

READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

For composite, porcelain or gold restorations. Super-Snap Buff Disks have no metal centers or exposed mandrels that can discolor or gouge composite materials. It features a synthetic velvet polishing cloth outer surface which retains polishing pastes with minimum splatter or waste.

PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Do not re-use Super-Snap Buff Disk.
- Mount a disk onto the metal mandrel provided securely.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective disk or mandrel i.e. deformation, dirt corrosion or surface flaw.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Use local dust extractor and protective equipment while using.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.
- Dispose in accordance with relevant local and national regulation.

STORAGE

Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.

STERILIZATION

Metal mandrel ●Perform autoclave under the condition described in Table B. ●When using intraorally, sterilize before each use. ●For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. ●Follow the instructions for use of washer disinfectant and sterilizer, respectively. ●Do not use chlorine disinfectant liquid. ●For the details about REPROCESSING procedures, visit www.shofu.de.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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