

Bossklein®

www.bossklein.com



V-WIPE ZERO

Surface Disinfectant Wipe

ALCOHOL FREE



100%
ALCOHOL
FREE

CONTACT
60 SEC



18gsm
EVERYDAY
WIPE

200
LARGE
WIPES

KEY FEATURES

Bossklein V-WIPE ZERO wipes are a highly effective cleaner and disinfectant. Designed primarily for the dental and medical sector. Use where time is critical, such as in between patients and in high traffic areas.

Ideal to use on the majority of non porous, wipeable surfaces; even those that are sensitive to alcohol, including dental chairs*

- Contact time of only 60 seconds
- 2 in 1; Cleans and disinfects
- Safe to use on sensitive surfaces*
- Modern, safe formulation
- Dual registration; Medical Device and Biocide
- Subtle apple aroma

ZERO has been rigorously tested to real world EN standards for proven effectiveness including Phase II Step I and II. Effective against a wide range of micro organisms (see table opposite) in only 60 seconds.

The revised formula takes advantage of modern, safer and greener chemical components. The new formulation contains a synergistic blend of two powerful biocides, is free from PHMB and contains detergent derived from renewable resources.

Bossklein V-WIPE ZERO is a 2 in 1 product. Containing a naturally sourced detergent to help clean and remove debris from a surface.

All V-WIPE Classic tubs are refillable and made from 35% recycled plastic (PCR), saving 31.5g of plastic for each tub. Each time a refill is used it saves 85% plastic compared to using a new tub.

The subtle apple aroma leaves surfaces and rooms reassuringly fresh, clean and ready for the next visitor.

COMPOSITION

Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecylmethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes (contain Amyl cinnamal, Limonene, Alpha-Isomethyl ionone).

INSTRUCTIONS FOR USE

1. Place refill bag in dispenser tub, 2. Cut open air tight plastic pouch across the top of the pouch, 3. Feed the centre wipe through the hole in the cap and replace, 4. Pull wipe at a 45° angle. Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action over 60 seconds. Allow surface moisture to dry or wipe with clean tissue. Contains detergent. For professional use only.

REGISTRATION

Dual registered as a Class IIa medical device for use on both non invasive medical devices and as a general purpose Biocide registered disinfectant.

TEST STANDARDS

The following test standards have been achieved: EN16777, EN16615, EN14885, EN14476, EN13727, EN13624.

VARIANTS AND PACKAGING

BOS2000 Tub of 200 Classic Wipes x 12 per box
BOS2001 Refill of 200 Classic Wipes x 12 per box
BOS2002 Kit of Classic Wipes - 6 x Refill + 1 x Tub

*Always test an inconspicuous area first. Do not use on copper or aluminium.

MICROBIOLOGICAL EFFICACY

| MICROORGANISM | CONTACT TIME |
|-----------------------------|---------------|
| Bactericidal | 60 sec |
| BS EN 16615:2015 | |
| BS EN 13727:2012 + A2:2015 | |
| Pseudomonas aeruginosa | 60 sec |
| Staphylococcus aureus | 60 sec |
| Enterococcus hirae | 60 sec |
| Yeasticidal | 60 sec |
| BS EN 16615:2015 | |
| BS EN 13624:2021 | |
| Candida albicans | 60 sec |
| Enveloped viruses | 60 sec |
| BS EN 14476:2013 + A2:2019 | |
| BS EN 16777:2018 | |
| Vaccinia virus | 60 sec |
| Hepatitis B virus (HBV) | 60 sec |
| Hepatitis C virus (HCV) | 60 sec |
| Hepatitis Delta virus (HDV) | 60 sec |
| HIV | 60 sec |
| Influenza virus | 60 sec |
| Coronavirus | 60 sec |
| Herpesviridae | 60 sec |
| Filoviridae (Ebola) | 60 sec |
| Rubella virus | 60 sec |
| Rabies virus | 60 sec |
| Measles virus | 60 sec |
| Poxviridae | 60 sec |

Wipes are available in our 'Classic' wipe 18gsm substrate, ideal for economic everyday use. Each wipe measures 200x200mm and each pack contains 200 Classic wipes. Both re usable dispenser tubs and economical refill packs are available.



Complete hygiene solutions

DirectaDentalGroup
DIRECTA topdental onsdent parkell KOHLER RONVIG polydentic
TrollDental Sordalim phytas CONTACTZ ceramic

Topdental (Products) Ltd
Unit 1, Holmfild Industrial Estate
Halifax, West Yorkshire HX2 9TN
Tel +44 (0)1535 652 750 www.topdental.co.uk

v1.2 0125