



## V-WIPE ZERO

### Surface Disinfectant Wipe

ALCOHOL FREE

**ULTRA**  
160 PREMIUM WIPES  
175 x 155mm



**100%**  
ALCOHOL  
FREE

**60 SEC**  
CONTACT



**40gsm**  
THICKER  
WIPE

**160**  
PREMIUM  
WIPES

#### KEY FEATURES

Bossklein V-WIPE ZERO wipes are a highly effective cleaner and disinfectant. Designed primarily for the dental and medical sector. Use where time is critical, such as in between patients and in high traffic areas.

Ideal to use on the majority of non porous, wipeable surfaces; even those that are sensitive to alcohol, including dental chairs\*

- Contact time of only 60 seconds
- 2 in 1; Cleans and disinfects
- Safe to use on sensitive surfaces\*
- Modern, safe formulation
- Dual registration; Medical Device and Biocide
- Subtle apple aroma

ZERO has been rigorously tested to real world EN standards for proven effectiveness including Phase II Step I and II. Effective against a wide range of micro organisms (see table opposite) in only 60 seconds.

The revised formula takes advantage of modern, safer and greener chemical components. The new formulation contains a synergistic blend of two powerful biocides, is free from PHMB and contains detergent derived from renewable resources.

Bossklein V-WIPE ZERO is a 2 in 1 product. Containing a naturally sourced detergent to help clean and remove debris from a surface.

All V-WIPE Classic tubs are refillable and made from 35% recycled plastic (PCR), saving 31.5g of plastic for each tub. Each time a refill is used it saves 85% plastic compared to using a new tub.

The subtle apple aroma leaves surfaces and rooms reassuringly fresh, clean and ready for the next visitor.

#### COMPOSITION

Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecyltrimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes (contain Amyl cinnamal, Limonene, Alpha-Isomethyl ionone).

#### INSTRUCTIONS FOR USE

1. Place refill bag in dispenser tub, 2. Cut open air tight plastic pouch across the top of the pouch, 3. Feed the centre wipe through the hole in the cap and replace, 4. Pull wipe at a 45° angle. Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action over 60 seconds. Allow surface moisture to dry or wipe with clean tissue. Contains detergent. For professional use only.

#### REGISTRATION

Dual registered as a Class IIa medical device for use on both non invasive medical devices and as a general purpose Biocide registered disinfectant.

#### TEST STANDARDS

The following test standards have been achieved: EN16777, EN16615, EN14885, EN14476, EN13727, EN13624.

#### VARIANTS AND PACKAGING

BOS3000 Tub of 160 Ultra Wipes x 12 per box  
BOS3001 Refill of 160 Ultra Wipes x 12 per box  
BOS3002 Kit of Ultra Wipes - 6 x Refill + 1 x Tub

\*Always test an inconspicuous area first. Do not use on copper or aluminium.

#### MICROBIOLOGICAL EFFICACY

MICROORGANISM	CONTACT TIME
<b>Bactericidal</b>	<b>60 sec</b>
BS EN 16615:2015	
BS EN 13727:2012 + A2:2015	
Pseudomonas aeruginosa	60 sec
Staphylococcus aureus	60 sec
Enterococcus hirae	60 sec
<b>Yeasticidal</b>	<b>60 sec</b>
BS EN 16615:2015	
BS EN 13624:2021	
Candida albicans	60 sec
<b>Enveloped viruses</b>	<b>60 sec</b>
BS EN 14476:2013 + A2:2019	
BS EN 16777:2018	
Vaccinia virus	60 sec
Hepatitis B virus (HBV)	60 sec
Hepatitis C virus (HCV)	60 sec
Hepatitis Delta virus (HDV)	60 sec
HIV	60 sec
Influenza virus	60 sec
Coronavirus	60 sec
Herpesviridae	60 sec
Filoviridae (Ebola)	60 sec
Rubella virus	60 sec
Rabies virus	60 sec
Measles virus	60 sec
Poxviridae	60 sec

Wipes are available in our 'Ultra' wipe 40gsm premium substrate, for ultimate strength and grip. Each wipe measures 175x155mm and each pack contains 160 Ultra wipes. Both reusable dispenser tubs and economical refill packs are available.



Complete hygiene solutions

DirectaDentalGroup  
DIRECTA topdental osmunda parkelli KOHLER RONWIG polydentica  
TrollDental Sendoline physics CONTACTEZ ceramir

Topdental (Products) Ltd  
Unit 1, Holmfield Industrial Estate  
Halifax, West Yorkshire HX2 9TN  
Tel +44 (0)1535 652 750 www.topdental.co.uk